

Smooth Muscle Actin (1A4)


Mouse anti-human Smooth Muscle Actin (SMA) Monoclonal Antibody (Clone 1A4)

REFERENCES AND PRESENTATIONS¹

- **ready-to-use** (manual or LabVision AutoStainer)
MAD-001195QD-3
MAD-001195QD-7
MAD-001195QD-12
- **Ready-to-use (MD-Stainer)²**
MAD-001195QD-3/V
MAD-001195QD/V
- **concentrated**
MAD-001195Q - 1:200 recommended dilution

COMPOSITION

Anti-human SMA mouse monoclonal antibody purified from serum and prepared in 10mM PBS, pH 7.4, with 0.2% BSA and 0.09% sodium azide

INTENDED USE : Immunohistochemistry (IHC) on paraffin embedded tissues. Not tested on frozen tissues or Western-Blotting

CLONE: 1A4

Ig ISOTYPE: Rabbit IgG_{2a/k}

IMMUNOGEN: N-Terminal decapeptide of alpha smooth muscle isoform of actin; acetylated at the N-terminus.

SPECIES REACTIVITY: In vitro diagnostics in humans. Not tested in other species

DESCRIPTION AND APPLICATIONS:

Actin is a major component of the cytoskeleton and is present in every cell type. Actin can be resolved on the basis of its isoelectric points into three distinctive components: alpha, beta, and gamma in order of increasing isoelectric point. Anti-Smooth Muscle Actin antibody does not stain cardiac or skeletal muscle; however, it will stain myofibroblasts and myoepithelial cells. This antibody could be used together with Muscle Specific Actin to distinguish leiomyosarcoma from rhabdomyosarcoma. In most cases of rhabdomyosarcoma, this antibody gives negative results whereas Muscle Specific Actin is positive in the rhabdomyoblasts. Leiomyosarcomas

¹ These references are for presentation in vials of Low Density Polyethylene (LDPE) dropper. In case the products are used in automated stainers, a special reference is assigned as follows:
- / L: Cylindrical screw-cap vials (QD-3 / L, QD-7 / L, QD-12 / L).
- / N: Polygonal screw-cap vials (QD-3 / N, QD-7 / N, QD-12 / N).
For different presentations (references / volumes) please contact the supplier.

² For Technical specifications for MD-Stainer, please contact your distributor.



are positive with both Muscle Specific Actin and Smooth Muscle Actin antibodies.

IHC POSITIVE CONTROL: Miometrium, colon

VISUALIZATION: Cell cytoplasm

IHC RECOMMENDED PROCEDURE:

- 4µm thick section should be taken on charged slides; dry overnight at 60°C
- Deparaffinise, rehydrate and HIER (heat induced epitope retrieval) – boil tissue in the Pt Module using Vitro S.A CITRATE buffer pH6³ for 20 min at 95°C. Upon completion rinse with 3-5 changes of distilled or deionised water followed by cooling at RT for 20 min
- Endogenous peroxidase block - Blocking for 10 minutes at room temperature using peroxidase solution (ref. MAD-021540Q-125)
- Primary antibody: incubate for 10 minutes [The antibody dilution (when concentrated) and protocol may vary depending on the specimen preparation and specific application. Optimal conditions should be determined by the individual laboratory]
- For detection use Master Polymer Plus Detection System (HRP) (DAB included; ref. MAD-000237QK)
- Counterstaining with haematoxylin and final mounting of the slide

STORAGE AND STABILITY:  Stored at 2-8°C. Do not freeze.  Once the packaging has been opened it can be stored until the expiration date of the reagent indicated on the label. If the reagent has been stored under other conditions to those indicated in this document, the user must first check its correct performance taking into account the product warranty is no longer valid.

WARNINGS AND PRECAUTIONS:

1. Avoid contact of reagents with eyes and mucous membranes. If reagents come into contact with sensitive areas, wash with copious amounts of water.
2. This product is harmful if swallowed.
3. Consult local or state authorities with regard to recommended method of disposal.
4. Avoid microbial contamination of reagents.

³ Ref: MAD-004071R/D



SAFETY RECOMMENDATIONS










This product is intended for laboratory professional use only. The product is NOT intended to be used as a drug or for domestic purposes. The current version of the Safety Data Sheet for this product can be downloaded by searching the reference number at www.vitro.bio or can be requested at regulatory@vitro.bio.

BIBLIOGRAPHY

- 1.Cooke P. A filamentous cytoskeleton in vertebrate smooth muscle fibers. J Cell Biol. 1976 Mar;68(3):539-56
- 2.Skalli O, Ropraz P, Trzeciak A, Benzonana G, Gillesen D, Gabbiani G. A monoclonal antibody against alpha-smooth muscle actin: a new probe for smooth muscle differentiation. J Cell Biol. 1986 Dec;103(6 Pt 2):2787-96
- 3.Gown AM, Vogel AM, Gordon D, Lu PL. A smooth muscle-specific monoclonal antibody recognizes smooth muscle actin isozymes. J Cell Biol. 1985 Mar;100(3):807-13

LABEL AND BOX SYMBOLS

Explanation of the symbols of the product label and box:

	Expiration date
	Temperature limit
	Manufacturer
	Sufficient content for <n> assays
	Catalog number
	Lot code
	Refer to the instructions of use
	Medical product for <i>in vitro</i> diagnosis.
	Material safety data sheet